



ASPIVENIN® PUMP ▾

INSECTS AND VENOMOUS SPECIES ▾



Always handy, **ASPIVENIN®** pump allows for a wait for the indispensable intervention of a doctor in serious cases (allergies or dangerous aggressions).

ASPIVENIN® pump in 4 steps :



Gently place the nozzle best-fitted to the type and location of the sting or bite, then arm the pump by pulling out the piston all the way (half for children).



With one hand, place **ASPIVENIN®** pump with its nozzle on the skin.



Fully depress the piston and you feel a powerful suction. Let **ASPIVENIN®** pump act 20 sec to 3 minutes (Adjust the suction time according to the stings or bites : see the instructions for use.) .



Then lift the piston button with the tip of the thumb to remove **ASPIVENIN®** pump.

Start again from step 2 if the pain has not disappeared.

We suggest you handle Aspivenin® several times with recommended instructions before use. We also advise you to keep your Aspivenin® within reach if you are in an environment at risk.

Duration of application :

The duration of application varies according to cases : for mosquito or fly stings, 20 to 30 seconds may be sufficient. For bee, wasp or horsefly, it is recommended to apply **ASPIVENIN®** pump for 1 to 3 minutes before cleaning the wound with an anti-septic.

Be careful : for bee stings, do not remove the sting before aspirating as its presence facilitates the extraction of venom. Remove it later.

For bites of a spider ou a scorpion, in order to avoid the inevitable coagulation of the wound, apply **ASPIVENIN®** pump for 1 to 2 minutes before cleaning the wound with an anti-septic.

For more serious stings, like that of a stone-fish, it is necessary to apply **ASPIVENIN®** during 2 to 3 minutes. Do not forget to regularly clean the wound with a disinfectant, and to wait for some rapid and indispensable medical care.